Instructions for use.  
Please read carefully!

## Safety for the denture **Protefix®** **Protefix® Active Cleanser**

**Protefix® Active Cleanser has an antibacterial effect, removes plaque and 99.99\* % of bacteria.**

\*(as per DIN EN 1040)

Scrupulous oral hygiene is required to remove food remnants which settle on the denture in just the same way as on natural teeth. If food remnants get under the denture, they may give rise to painful inflammations since the natural oral flora is disturbed whilst denture are being worn and it is impossible for the oral cavity to clean itself completely. So in this case you should not depend on your toothbrush alone but rather on a good cleaning tablet.

**Protefix® Active Cleanser** is particularly effective in this case since it has both excellent cleaning power and a strong deodorising effect. Due to its make-up **Protefix® Active Cleanser** has sufficient active oxygen to clean the denture independently, thoroughly and quickly without attacking the denture material.

### **Dosage and mode of application:**

Dissolve one **Protefix® Active Cleanser** tablet

in a glass of lukewarm water (100-200 ml, about 35° C) and place the denture or brace immediately in the glass. The denture or brace should be completely covered by the cleaning solution. Clean and fresh in 3 minutes, disinfected in 10 minutes. Cleaning also possible overnight. Remove the denture and rinse well under running water before replacing it in the mouth.

### **Please note the following important advice:**

Apply only outside of the mouth. Avoid contact with eyes. The product should be stored out of the reach of young children. Should a tablet or cleaning solution swallowed by mistake, please immediately seek medical advice. Store in a cool and dry place.

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Queisser Pharma GmbH & Co. KG
Helago-Pharma GmbH